



BEYOND A BETTER BOX™

XW™-100 AUTOMATED HEMATOLOGY ANALYZER
CLIA WAIVED
SHUTDOWN QUICK GUIDE



XW-100 SHUTDOWN

Shutdown

Shutdown of the XW-100 is not required. If your laboratory would like to shutdown the instrument, please refer to the instructions below.

1. Press the [↵] button in the upper right corner of the Main Screen; a second screen displays.
2. Press the [Move Instrument] button; this will cause the reagents to drain and the XW-100 to shut down.
3. Follow the on-screen instructions; you may be prompted to press the Power switch on the right side of the analyzer to complete Shutdown.

NOTE: Once shutdown has been performed:

1. The internet connection will need to be reestablished.
2. New reagents will need to be scanned to replace existing reagents.

NOTES



Sysmex America, Inc.
577 Aptakistic Road
Lincolnshire, IL 60069, U.S.A.
Tel. +1 (800) 379-7639
www.sysmex.com/us

Sysmex Corporation
1-5-1 Wakinohama-Kaigandori
Chuo-ku, Kobe 651-0073, Japan